

# Product Specifications

## Design

- Microprocessor-based system architecture
- Modular system consisting of display/control module and pneumatic drive unit
- Proprietary deflation timing management

## Electrical

- AC requirements:
  - 90–264 VAC 47–63 Hz
- Typical power consumption: 245 watts
- Maximum power consumption: 420 watts
- Battery operating time:
  - 90 minutes minimum with full charge
  - 180 minutes with optional second battery
- Typical battery recharging time:
  - 80% in 4 hours from full discharge
  - Recharge to 80% indicated by yellow light

## Mechanical Dimensions

- Control module with monitor:
  - 10" high (25.4 cm) x 13.75" wide (35 cm) x 2" deep (5 cm)
- Pneumatic drive unit:
  - 31.5" high (80 cm) x 13.5" wide (34.3 cm) x 21" deep (53.3 cm)

## Mechanical Weight

- Control module:
  - 5 lbs (2.3 kg)
- Pneumatic unit for AutoCAT®2:
  - 94 lbs (41.7 kg)
- Total weight for AutoCAT2:
  - 99 lbs (44.0 kg)
- Total weight for AERO™ Series:
  - 90 lbs (40.0 kg)

## Pneumatics

- Drive system: Stepper motor-driven bellows
- Drive gas: USP-grade helium
- Helium tank:
  - Disposable canister (500 psi) or refillable (2000 psi) cylinder—US Approval; (2900 psi) cylinder—European Approval
- Pumping volume:
  - 0.5cc to 50cc, adjustable in 0.5cc increments
- Counterpulsation rate: 40 to 200 pulsations/minute
- Assist ratio options

## Condensation Removal

- Thermoelectric system removes moisture continuously from pneumatic system without interrupting counterpulsation

## System Modes

- AutoPilot™:
  - Automatically selects ECG/AP signal, sources, trigger mode, and timing method as well as timing settings
  - Automatically changes settings to optimize assist
  - Proprietary software sets timing to correspond to individual patient needs
- Operator:
  - Allows user control of most pump functions

## Trigger Modes

- ECG (PATTERN, PEAK, AFIB):
  - Microprocessor-based R-waveform trigger detection algorithms
- Pacer (VPACE, APACE):
  - Low level (skin) ECG input
    - Pulse width 0.1 to 0.5 ms and pulse amplitude => +5 to +700 mV
    - Pulse width => 0.5 to 2 ms and pulse amplitude => +2 to +700 mV
  - High level (monitor) input
    - Pulse width 0.1 to 2 ms and pulse amplitude => 1 V
    - AV pacer detection is <250 msec between pacer pulses
- Arterial pressure (AP):
  - Microprocessor-based waveform trigger detection algorithm
- Internal:
  - Default to 80 bpm; adjustable 40 to 120 bpm
- Filtering:
  - Diathermy, 30 Hz low pass

## General Trigger Selection Criteria (AutoPilot Mode)

### ECG TRIGGER MODES:

- PATTERN: HR <130 bpm no arrhythmia
- PEAK: HR >130 bpm or arrhythmia detected and arrhythmia timing OFF\*
- AFIB: Any HR with arrhythmia detected\*
- VPACE: Single or dual pacer (<250 msec apart) and no QRS or AP waveform detected
- APACE: Single pacer with R-wave >100 msec later. Transition only

### AP TRIGGER MODE:

- No ECG signal or noisy ECG signal

\*Based upon deflation timing management.

## Inflation/Deflation Timing Methods

### INFLATION TIMING METHODS:

- Predictive: AP waveform analysis to set inflation
- Weissler: ECG only, inflation timing based on systolic time intervals

### DEFLATION TIMING METHODS:

- R-wave: Real-time deflation on R-wave
- Predictive: Deflation set to occur just prior to next systolic rise
- Weissler: ECG only, deflation timing based on diastolic intervals

### MANUAL:

- User set inflation and deflation timing in Operator Mode

## Inflation/Deflation Timing Limits (Operator Mode)

- ECG:
  - Inflation, 20%–80% of R-R interval
  - Deflation, 30%–120% of R-R interval
- AP:
  - Inflation, 0–35% of peak systole-peak systole interval
  - Deflation, 35%–75% of peak systole-peak systole interval
- AFIB Trigger Mode:
  - Inflation 80 to 430 ms after R-wave trigger event
  - Deflation on R-wave

## Display

- Type:
  - Color LCD flat screen
- Channels:
  - Three-channel multicolor waveforms
    - ECG: Green trace with white highlight on assisted portion
    - AP: Red trace calibrated for direct reading of AP, white highlight on assisted portions when in Operator Mode
    - Balloon pressure: Blue trace calibrated in mm Hg and displayed continuously
- Timing reference display:
  - Numerical timing settings in both operating modes as well as a bar graph displaying inflate/deflate events in Operator Mode
- Cursor:
  - Measurement of AP and balloon pressure waveforms

## Ordering Information

Order No.	Description
IAP-0400	<b>AutoCAT 2</b> 1 (IABP) System includes: <ul style="list-style-type: none"><li>• Proprietary timing software</li><li>• AutoPilot operational mode</li></ul>
IAP-0435	<b>AERO Series–AutoCAT 2</b>

The products above are also available in multiple languages. Contact Arrow for availability.

## Alphanumeric Data

- Patient hemodynamics: Heart rate, AP—systolic, augmented, diastolic, and mean arterial. When in 1:2 or lower assist ratio the assisted values are displayed in white and the unassisted values are displayed in yellow
- Displayed parameters: ECG source and gain state, alarm status with timer, ON Battery indication, operation mode selection, AP alarm parameter and limit, timing settings, helium tank level, arrhythmia detection, and timing status
- Operations status: Operational mode, trigger mode, helium tank gauge, alarm/battery charge status, balloon volume
- Diagnostic alarm/help messages: Preprogrammed troubleshooting prompts/help

## Strip Chart Recorder

- Recorder:
  - Dual-channel dot matrix: Dot density 400 dots/inch, 25 mm/s
- Waveforms:
  - ECG, AP, or balloon pressure (one or two recorded)
- Alphanumeric:
  - Operational mode, trigger mode, ECG lead/source, AP source, AP alarm status, timing settings, assist ratio, balloon volume, timing method, arrhythmia status, alarm condition, date, time, patient hemodynamics

## Display Freeze

- Freezes approximately 7 seconds of patient data on screen

## Patient Signal Inputs

- ECG:
  - 5 lead skin cable (I, II, III, aVR, aVL, aVF and V)
  - High level monitor input (0 to 5 V)
- AP:
  - AP transducer (Spectramed or equivalent), 50 mV/V/cm Hg
  - High-level monitor input (1 V = 100 mm Hg)

Note: Additional system specifications are available from Arrow upon request. Specifications are subject to change without notice.

Caution: U.S. Federal law limits this device to sale by or on order of a physician.

U.S. Patent Nos. 6,258,035, 6,569,103, 6,887,206, and 5,913,814; additional patents pending.

### Distribution Worldwide:

Arrow offices are located in Belgium, Canada, Czech Republic, France, Germany, Greece, Hungary, India, Italy, Japan, Mexico, Netherlands, Slovakia, South Africa, Spain, United Kingdom, and the United States.

Arrow International, Inc., 2400 Bernville Road, Reading, PA 19605 U.S.A. Tel: 610-378-0131  
[www.proactivecounterpulsation.com](http://www.proactivecounterpulsation.com)

A2-SPEC      8/07      CRB-07-0225

© 2007 Arrow International, Inc. U.S.A.

All rights reserved.

Printed in the U.S.A.

