

Product Specifications

Design

- FiberOptix™ capable:
 - AP signal transmitted at speed of light
- Proprietary WAVE® algorithm
- Proprietary Aortic Flow Timing Method
- AutoPilot™ mode of operation
- Microprocessor-based system architecture
- Modular system consisting of display/control module and pneumatic drive unit
- Proprietary deflation timing management

Electrical

- AC requirements:
 - 90–264 VAC 47–63 Hz
- Typical power consumption: 245 watts
- Maximum power consumption: 420 watts
- Battery operating time:
 - 90 minutes minimum with full charge
 - 180 minutes with optional second battery
- Typical battery recharging time:
 - 80% in 4 hours from full discharge
 - Recharge to 80% indicated by yellow light

Mechanical Dimensions

- Control module with monitor:
 - 10" high (25.4 cm) x 13.75" wide (35 cm) x 2" deep (5 cm)
- Pneumatic drive unit:
 - 31.5" high (80 cm) x 13.5" wide (34.3 cm) x 21" deep (53.3 cm)

Mechanical Weight

- Control module:
 - 5 lbs (2.3 kg)
- Pneumatic unit for AutoCAT 2 WAVE®:
 - 95.5 lbs (42.4 kg)
- Total weight for AutoCAT 2 WAVE:
 - 100.5 lbs (44.7 kg)
- Total weight for AERO™ Series:
 - 91.5 lbs (40.7 kg)

Pneumatics

- Drive system: Stepper motor-driven bellows
- Drive gas: USP-grade helium
- Helium tank:
 - Disposable canister (500 psi) or refillable (2000 psi) cylinder—US Approval; (2900 psi) cylinder—European Approval
- Pumping volume:
 - 0.5cc to 50cc, adjustable in 0.5cc increments
- Counterpulsation rate: 40 to 200 pulsations/minute
- Assist ratio options

Condensation Removal

- Thermoelectric system removes moisture continuously from pneumatic system without interrupting counterpulsation

System Modes

- AutoPilot:
 - Automatically selects ECG/AP signal, sources, trigger mode, and timing method as well as timing settings
 - Automatically changes settings to optimize assist
 - Proprietary software sets timing to correspond to individual patient needs
- Operator:
 - Allows user control of most pump functions

Trigger Modes

- ECG (PATTERN, PEAK, AFIB):
 - Microprocessor-based R-waveform trigger detection algorithms
- Pacer (VPACE, APACE):
 - Low level (skin) ECG input
 - Pulse width 0.1 to 0.5 ms and pulse amplitude => +5 to +700 mV
 - Pulse width => 0.5 to 2 ms and pulse amplitude => +2 to +700 mV
 - High level (monitor) input
 - Pulse width 0.1 to 2 ms and pulse amplitude => 1 V
 - AV pacer detection is <250 msec between pacer pulses
- Arterial pressure (AP):
 - Microprocessor-based waveform trigger detection algorithm
- Internal:
 - Default to 80 bpm; adjustable 40 to 120 bpm
- Filtering:
 - Diathermy, 30 Hz low pass

General Trigger Selection Criteria (AutoPilot Mode)

ECG TRIGGER MODES:

- PATTERN: HR <130 bpm no arrhythmia
- PEAK: HR >130 bpm or arrhythmia detected and arrhythmia timing OFF*
- AFIB: Any HR with arrhythmia detected*
- VPACE: Single or dual pacer (<250 msec apart) and no QRS or AP waveform detected
- APACE: Single pacer with R-wave >100 msec later. Transition only

AP TRIGGER MODE:

- No ECG signal or noisy ECG signal

*Based upon deflation timing management.

Inflation/Deflation Timing Methods

INFLATION TIMING METHODS:

- Aortic Flow:
 - Proprietary WAVE algorithm sets the timing intra-beat on average 1 ms of aortic valve closure¹
- Predictive: AP waveform analysis to set inflation
- Weissler: ECG only, inflation timing based on systolic time intervals

DEFLATION TIMING METHODS:

- R-wave: Real-time deflation on R-wave
- Predictive: Deflation set to occur just prior to next systolic rise
- Weissler: ECG only, deflation timing based on diastolic intervals

MANUAL:

- User set inflation and deflation timing in Operator Mode

Inflation/Deflation Timing Limits (Operator Mode)

- ECG:
 - Inflation, 20%–80% of R-R interval
 - Deflation, 30%–120% of R-R interval
- AP:
 - Inflation, 0–35% of peak systole-peak systole interval
 - Deflation, 35%–75% of peak systole-peak systole interval
- AFIB Trigger Mode:
 - Inflation 80 to 430 ms after R-wave trigger event
 - Deflation on R-wave

Display

- Type:
 - Color LCD flat screen
- Channels:
 - Three-channel multicolor waveforms
 - ECG: Green trace with white highlight on assisted portion
 - AP: Red trace calibrated for direct reading of AP, white highlight on assisted portions when in Operator Mode
 - Balloon pressure: Blue trace calibrated in mm Hg and displayed continuously
- Timing reference display:
 - Numerical timing settings in both operating modes as well as a bar graph displaying inflate/deflate events in Operator Mode
- Cursor:
 - Measurement of AP and balloon pressure waveforms

Ordering Information

Order No.	Description
IAP-0500	AutoCAT2 WAVE 1 (IABP) System includes: <ul style="list-style-type: none"> • FiberOptix Sensor Technology • WAVE algorithm: Physiologically based, proprietary timing algorithm • Aortic flow timing method • AutoPilot mode of operation
IAP-0535	AERO Series—AutoCAT2 WAVE

The products above are also available in multiple languages. Contact Arrow for availability.

Alphanumeric Data

- Patient hemodynamics: Heart rate, AP—systolic, augmented, diastolic, and mean arterial. When in 1:2 or lower assist ratio the assisted values are displayed in white and the unassisted values are displayed in yellow
- Displayed parameters: ECG source and gain state, alarm status with timer, ON Battery indication, operation mode selection, AP alarm parameter and limit, timing settings, helium tank level, arrhythmia detection, and timing status
- Operations status: Operational mode, trigger mode, helium tank gauge, alarm/battery charge status, balloon volume
- Diagnostic alarm/help messages: Preprogrammed troubleshooting prompts/help

Strip Chart Recorder

- Recorder:
 - Dual-channel dot matrix: Dot density 400 dots/inch, 25 mm/s
- Waveforms:
 - ECG, AP, or balloon pressure (one or two recorded)
- Alphanumeric:
 - Operational mode, trigger mode, ECG lead/source, AP source, AP alarm status, timing settings, assist ratio, balloon volume, timing method, arrhythmia status, alarm condition, date, time, patient hemodynamics

Display Freeze

- Freezes approximately 7 seconds of patient data on screen

Patient Signal Inputs

- ECG:
 - 5 lead skin cable (I, II, III, aVR, aVL, aVF and V)
 - High level monitor input (0 to 5 V)
- AP:
 - Fiber optic signal input from LightWave™ IAB Catheter (WAVE)
 - AP transducer (Spectramed or equivalent), 50 mV/V/cm Hg
 - High-level monitor input (1 V = 100 mm Hg)

Note: Additional system specifications are available from Arrow upon request. Specifications are subject to change without notice.

Caution: U.S. Federal law limits this device to sale by or on order of a physician.

U.S. Patent Nos. 6,258,035, 6,569,103, 6,887,206, and 5,913,814; additional patents pending.